


User Manual

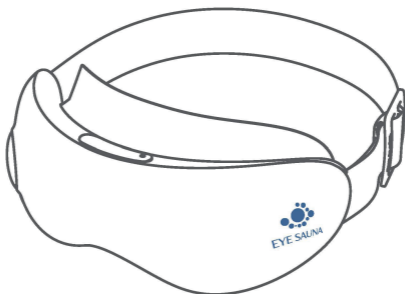

Eye Sauna™

IMPORTANT SAFETY INSTRUCTIONS

1. Please consult a doctor before use if you fit into any of the following user groups:
Persons who have undergone recent eye surgery or eye conditions such as glaucoma, cataracts, retinal detachment.
The elderly or persons who are pregnant / postpartum.
Persons allergic to silicone material.
Persons with skin infections, sensitivity, redness or burns on the skin.
Persons with any swelling or wounds around the eyes.
Persons with a heart condition, malignant tumours, or other serious health conditions.
Persons who are receiving medical treatment.
Persons with high or abnormal blood pressure.
2. If you experience any discomfort from the warm mist during use, please stop using the device immediately.
3. This device is intended for personal use only and not for commercial purposes.
4. The device is intended for persons over 18 years of age.
5. Do not submerge the device in water. Avoid contact with sharp objects.
6. Never put your fingers into the mist outlets to avoid contact with heating plates. These are hot and may burn.
7. The device contains batteries. Do not use in high temperatures or humid environments.
8. Only use the supplied charging cable with a DC 5V 1A charger (not included) to recharge the device.
9. Do not use the device while driving, walking or other activities where you may risk personal injury.
10. Do not use over sensitive skin areas or on persons with poor circulation.
11. Do not wear contact lenses when using this device.
12. Do not use the device if you have keratoconus or corneal grafts.

SAVE THESE INSTRUCTIONS

Electrical Rating:

Model	ES1	Eye Sauna.com.au
Input Rating	5VDC 1A	PH: 07 xxxx-xxxx
Battery	3.7VDC, 1500mAh	
Power	5W	
IP Rating	IPx0	

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

Children should be supervised to ensure that they do not play with the appliance.

Power supplied to the Eye Sauna must be extra low voltage corresponding to the marking on the appliance.

Instructions are also available on our website.

ABOUT YOUR EYE SAUNA™

1. What's in the Box

- 1 x Eye Sauna
- 1 x Cleaning Brush
- 1 x Water Filling Bottle (10ml)
- 1 x Charging Cable
- 1 x User Manual

2. Specifications

- Model: Eye Sauna
- Input: 5V = 1A
- Rated Power: 5W
- Battery Capacity: 1500mAh
- Water Tank Capacity: 5ml

Note: Your Eye Sauna™ can be used once or twice per day.

CHARGING YOUR EYE SAUNA™

Fully charge the device before the first use. Use the provided charging cable and a DC 5V 1A charger (not included) for charging.

Note:

- Do not use the device while charging.
- It takes 3 hours to fully charge the device.

ADDING LIQUID TO THE WATER TANK

Before using the device, fill the water tank with liquid.

- a. Open the water tank lid gently.
- b. Add liquid such as distilled water or filtered potable tap water to the tank and close the lid. Do not add oily, viscous or corrosive liquids.

Note:

It is recommended to use the provided small water bottle when adding purified water or distilled water to the tank. Empty the water tank after each use and keep it dry to avoid bacteria.

BEFORE USING YOUR EYE SAUNA™

- a. Remove any eye accessories (e.g., eyeglasses, false eyelashes etc.) and heavy makeup. Remove contact lenses.
- b. Wear the device and make sure it fits your eyes properly. Adjust the elastic band to fit your head. (Ensure the silicone cushion is placed securely before wearing.)

Use the device with your eyes open. Blink normally. You can also close your eyes at times if you want.

CHARGING YOUR EYE SAUNA™

Elastic Band

Water Tank Lid



Power Button / Power Indicator

Mist Outlets

Charging Port

Water Tank / Water Tank Light

Detachable & Washable Silicone Cushion

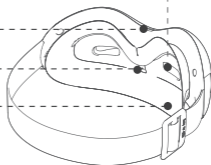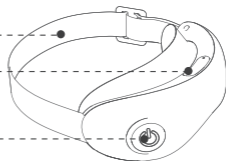



Power Button

a. Press and hold for 2 seconds to turn on/off the device.

Eye Sauna will start preheating mode for 4 minutes. Device will alert you with audible message when it is ready to use. After this, place device on face and adjust the strap for comfort.

b. Short press to cycle through 3 modes.

M 1: Continuous Hot Mist (auto-off timer:11min)

Recommended setting.

M 2: Interval Hot Mist (auto-off timer: 11min)

M 3: Cold Mist (Room temperature, auto-off timer: 11min)

c. Double press to switch between voice mode and silent mode.

In silent mode, the voice broadcast and built-in music will be disabled. The Bluetooth music will be paused but can be resumed via your smartphone.

d. Device will turn off when treatment cycle is completed.

CONNECTING TO BLUETOOTH

- a. Turn on the device and enable Bluetooth on your smartphone.
- b. Search for "Eye Sauna" via your Bluetooth settings and connect to play your music playlist through the device.

Note: Once paired successfully, the device will be automatically connected to Bluetooth next time, followed by a voice prompt "Bluetooth connected".

4. Indicator / Light Introduction

Status	Indicator / Light
Device is on / Bluetooth disconnected	Power indicator turns solid white. Water tank light illuminates and then turns off after 5 seconds.
Bluetooth connected	Power indicator turns solid blue.
Low battery / Charging	Power indicator flashes red slowly.
Fully charged	Power indicator turns solid red.
Charging at a voltage higher than 5V	Power indicator keeps flashing red quickly for 1 minute and then turns off, indicating an improper charger is used.

CLEANING AND MAINTENANCE

1.Device:

- Turn off the device or unplug the charging cable and allow the device to cool down before cleaning or storing.
- Wipe the surface with a soft cotton cloth. You may like to use a cloth slightly dampened with mild detergent. Do not use abrasives, solvents, chemical cleaners, or harsh detergents.
- Do not expose the device to direct sunlight or high temperatures. If the device has not been used for over 3 months, please charge it from time to time to prolong its lifespan.
- It is recommended to dry the device after each use to prolong its lifespan and discourage mould.
- Clean the water tank with the provided cleaning brush if necessary.
- Do not disassemble the device by yourself, as this may cause damage and void the manufacturer's warranty.

2.Silicone cushion:

- a. You can remove the silicone cushion from the device and rinse with water or hand wash with soap or mild detergent. Never use abrasives, solvents, chemical cleaners, or harsh detergents.
- b. Dry with a soft cotton cloth. Ensure the silicone cushion is completely dry before attaching it back to the device.
- c. To avoid size enlargement or breakage, never pull the silicone cushion violently.

3.Atomizer plates:

- a. To avoid bacteria and clogging of the atomizer plates, empty water tank after each use.
- b. When the mist output is significantly reduced, add equal amounts of white vinegar and water to the water tank and let it stand for 3 minutes before emptying. Rinse the tank at least twice with water and dry the whole device. Repeat this process if required.

TROUBLESHOOTING

Problem	Possible Cause	Solution
No mist or less mist.	No or little liquid in the water tank.	Ensure there is enough liquid in the water tank before use.
	Using water with high mineral content	Use demineralised water or filter potable tap water.
	The atomizer plates are clogged.	Refer to point 3 in the "CLEANING AND MAINTENANCE" part.
The mist feels cold.	"Cold Mist" is selected.	Press the Power Button to switch to "Continuous Hot Mist" or "Interval Hot Mist".
	The device needs time to heat the mist up.	Wait for the device to finish preheating mode.
	The built-in overheat protection is activated.	It is normal. The device will automatically restart after cooling down.

Problem	Possible Cause	Solution
Power-on failure.	Out of power.	Charge the device.
	Short press the Power Button.	Long press the Power Button to turn on the device.
The device shuts off suddenly.	Out of power.	Charge the device.
	The built-in auto-off timer is activated.	Restart the device if needed.
Charging failure.	Using other types of charging cables.	Use the provided Type-C charging cable only.
	Using a charger with voltage higher than 5V.	Use a DC 5V 1A charger.
Bluetooth connection failure.	Bluetooth on the mobile device is not enabled.	Enable Bluetooth on your mobile device and connect to "Eye Sauna".
	The device is connected to another mobile device.	Ensure the device is fully disconnected from another mobile device.
	The device is off.	Turn on the device and connect to Bluetooth.
No music or voice broadcast.	The silent mode is selected.	Double press the Power Button to switch to voice mode.
	Bluetooth music is playing at volume level 0.	Adjust the volume level through your mobile device.